

# Omnicell Medication Adherence Catalogue

The UK's No.1 choice for medication adherence packaging






# Contents

## SureMed packaging

Pill packs and care packs - our card range	4
Solutions for the card range: Blisters	10
Custom cards	13
Accessories for our card range	14
Pill packs and care packs - our clear range	16
Monthly care pack kit	18
Assembling your pack	19
Pill pack and care pack solutions	20
Stickers and labels	24

## De-blistering Solutions

DB500	26
FREE weekly pill pack point of sales toolkit	27

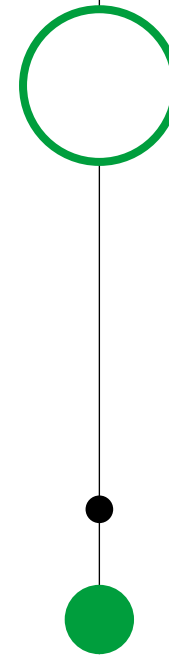

SureMed by Omnicell

# card & clear packaging

Weekly pill packs

Monthly care packs

Blisters



SureMed by Omnicell adherence packs will help your patients take the right pills and right dose at the right time. It will also save you time and money. Our SureMed by Omnicell packs include a range of sizes, so you can pack larger quantities of medication. Our weekly pill packs are available in both a clear and card range so you can choose the solution that is best for you and your patients.

We pride ourselves on the fact that our solutions make taking medicine easy. Each personal pack contains the right pills and dose, ready to take at the right time. Contact us today and we'll support you in your discussions with your customers.

## Quality you can trust.

- Our card and clear products are rigorously tested and carry the Class B quality standard.\*
- Class B certification means that our cards and blisters are thoroughly and independently tested to ensure compliance with USP for moisture, this ensures the integrity of the package and stability of the medication is not compromised.
- Without the right packaging, drugs can be damaged in transit or exposed to contaminants like moisture that limit their effectiveness.
- Don't take the risk - using cheaper medication adherence (MA) packaging could have worrying consequences for your patients' health and wellbeing.

# A commitment to sustainable consumable products (UK)

As an industry leader, Omnicell takes environmental sustainability seriously.

We work to ensure that we operate with a commitment to continuous improvement in environmental sustainability through reducing, recycling and reusing of our consumable product range.

We ensure in order to meet this need, the plastics in our blister cards are:

- Formed of at least 80% recycled plastic
- 100% recyclable (when separate from the card)



For any further enquiries regarding the environmental impact of Omnicell products, visit [www.omnicell.co.uk](http://www.omnicell.co.uk) or contact [orderuk@omnicell.com](mailto:orderuk@omnicell.com)

## Weekly pill pack card range & plastic template

- We have integrated a plastic regrind process into our supply chain.
- Our blister cards are produced with plastic that's composed of up to 80% of this reground plastic
- The plastic template is made from recyclable material and can be fully recycled.

## Weekly pill pack clear range

- The plastic tray is made from recycled materials.
- The plastic tray and paper insert is full recyclable.

# Our card range

## Pill packs and care packs

We provide a variety of different pill packs in our card range for pharmacies to offer. Our weekly pill packs are designed specifically for community patients and our monthly care packs are for use in care homes.

We also offer our popular weekly flexi pack with simple tear and go doses, our weekly max pack for patients with more complex medication regimes and our monthly care pack book which is split into seven days of single dose over a four week period - ideal for care home patients.

Whatever your requirements, we have a design to suit you and your patients. You can even put your own brand on our packs to promote your pharmacy and services to your customers. All our packs are excellent quality, easy to use and disposable.


**Please note:** Our entire range of card based weekly pill packs (excluding weekly pill pack flip) are supplied complete with card, blister and filling template unless otherwise stated.

## Monthly care packs

Our card range of 28 day, single-dose packs designed specifically for use in care homes.

### monthly care pack book

Requires blisters and templates.  
Book style assembly.

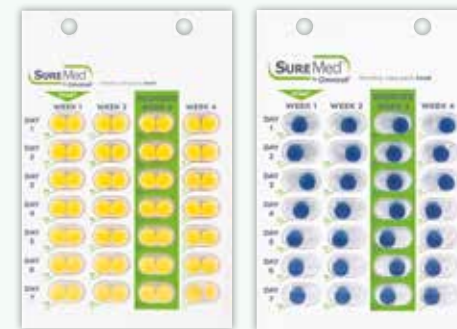

heat seal

Qty: 500 units

Code: 3300-01

pressure seal

Qty: 250 units

Code: 3300-02

pressure seal (bottom label)

Qty: 250 units

Code: 3300-12

### monthly care pack flip

Requires blisters and templates.  
Flip style assembly.



pressure seal

Qty: 250 units

Code: 3300-07

## Weekly pill packs



### weekly pill pack

pressure-seal

Pack includes sealing template

Our most popular 7 day, 4 dose pill pack.

Available to custom design (Min order of 100 Boxes)

Qty: 250 units

Code: 3301-26A PACK



### weekly pill pack max

pressure-seal

Pack includes sealing template

A large version of the weekly pill pack for patients taking a large number of medications.

Available to custom design (Min order of 100 Boxes)

Qty: 100 units

Code: 3301-45A PACK



### weekly pill pack flexi

Allows patients or carers to tear off individual doses and carry either a single dose, or doses for the whole day.

Available to custom design (Min order of 100 Boxes)

pressure-seal (left to right)

Pack includes sealing template

Qty: 250 units

Code: 3315-10A-PACK

pressure-seal (right to left)

Pack includes sealing template

Qty: 250 units

Code: 3315-11A-PACK



### weekly pill pack duo


pressure-seal

Pack includes sealing template

A pack specifically designed for patients taking medication twice a day.

Qty: 250 units

Code: 3301-50A-PACK

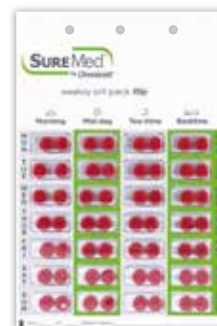

### weekly pill pack flip

pressure-seal

Requires blisters and templates.  
Flip style assembly.

Qty: 250 units

Code: 3300-06

# Assembling the card packs



## 1: Peel

Remove the pressure sensitive liner from right to left to reveal the sealing surface. Put the card on the filling template (glue side up). The rest of the liner must be left attached to the foil side until you've completed dispensing.



## 2: Insert

Once the card is on the filling template, immediately insert the blister sheet into the card starting with the top row. Lower the rest of the sheet into the remaining holes. Apply the roller to ensure the blister sheet is securely held by the adhesive to the card.



## 3: Dispensing

Using the reverse side as a guide, fill the appropriate blister with the correct medication.



## 4: Remove Liner

Only when you are ready to seal the pack remove the pressure sensitive liner.



## 5: Fold

As soon as the liner is removed, fold the sealing surface to cover the filled blister.



## 6: Seal

Using the roller apply firm pressure to the whole surface to seal. Place the correct patient medication labels on the instruction flap. Indicate the shape, colour etc. of medication under each label that is applied. The SureMed by Omnicell pack is now complete.

## TOP TIP:

For the perfect seal always place the filling template on a flat, even surface and remember to apply pressure using a roller!

If medication changes are needed, sealed packs can be corrected by using pill pack repair patches - see page 15

- 1: Carefully pierce and remove the paper from the rear of the blister that needs to be changed.
- 2: Amend the medication and check as appropriate.
- 3: Carefully reseal the blister using a pill pack repair patch. The adhesive edge around the patch should be firmly pressed over the rear edge of the blister cavity to form a new seal. This enables medication changes to be made quickly before the pack is handed to the patient and without the need for new packs to be assembled.

Pill pack patches can be purchased from Omnicell



Code: PLUSPAK-PATCHES





# Fully compatible with automated machines such as the VBM 200F

Cards can display coloured med images and coloured time passes.  
Instructions can be printed on cards to enhance ease-of-use.  
Different formats of packs are available meet the needs of the patient.

## VBM supporting Items

[SureMed by Omnicell VBM card with adhesive label](#)

Code: 3301-47D

[SureMed by Omnicell VBM standard card](#)

Code: 3301-48

[SureMed by Omnicell VBM print direct to card](#)

Code: 301-96GLB

# Solutions for the card range

## Blisters

The majority of our pill packs come with blisters already included but they need to be ordered separately for use with the weekly pill pack flip, monthly care pack book and monthly care pack flip.


These are available in a range of sizes.

### Weekly Blisters

#### weekly pill pack *flip* blisters - small

Qty: 500 units


Code: 200-47-MP

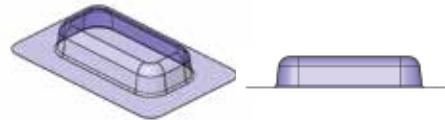

#### weekly pill pack *flip* blisters - large

Qty: 500 units


Code: 200-49-MP

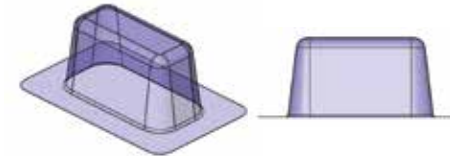

#### weekly pill pack *flip* blisters - standard

Qty: 500 units


Code: 200-48-MP

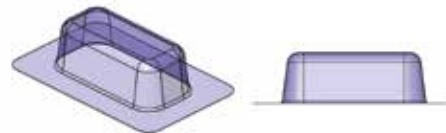

#### weekly pill pack *flip* blisters - extra large

Qty: 500 units

Code: 3301-26A PACK

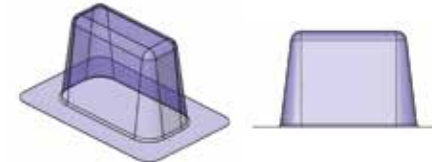


## Monthly Blisters

monthly care pack *book* blisters - small

Qty: 1000 units


Code: 200-15-MP

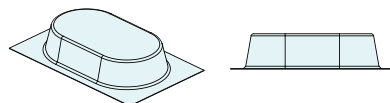

monthly care pack *flip* blisters - small

Qty: 500 units


Code: 200-47-MP

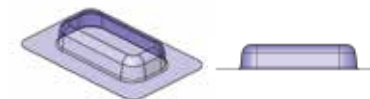

monthly care pack *book* blisters - medium

Qty: 1000 units


Code: 200-16-MP

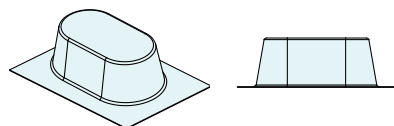

monthly care pack *flip* blisters - medium

Qty: 500 units


Code: 200-48-MP

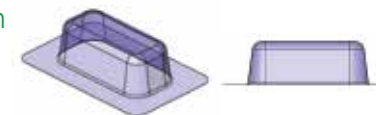

monthly care pack *book* blisters - large

Qty: 1000 units


Code: 200-18-MP

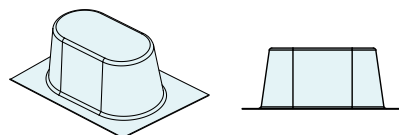

monthly care pack *flip* blisters - large

Qty: 500 units


Code: 200-49-MP

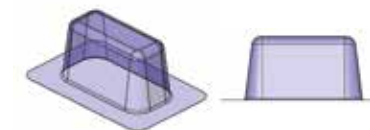

monthly care pack *book* blisters - extra large

Qty: 1000 units


Code: 200-50-MP

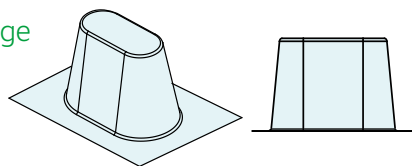

monthly care pack *flip* blisters - extra large

Qty: 500 units

Code: 200-49J-MP

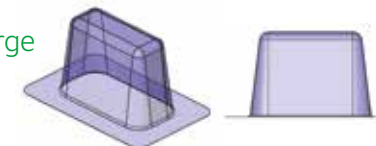

# Solutions for the card range


## Extra Blisters

The majority of our pill packs come with blisters already included. You can purchase extra blisters for all of our card packs.

### extra weekly pill pack blisters - standard

Qty: 250 units


Code: 200-46-MP

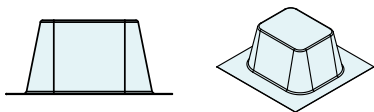

### extra weekly pill pack - duo blisters

Qty: 250 units


Code: 200-53-MP

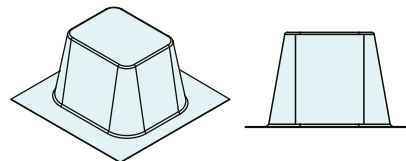

### extra weekly pill pack blisters - large

Qty: 250 units


Code: 200-56-MP

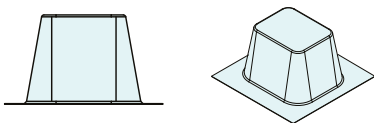

### extra weekly pill pack - flexi blisters

Qty: 250 units



Code: 205-28-MP

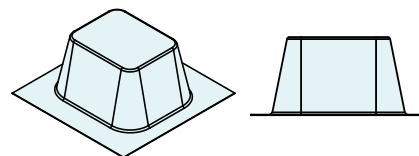

### extra weekly pill pack - max blisters

Qty: 250 units

Code: 200-65-MP

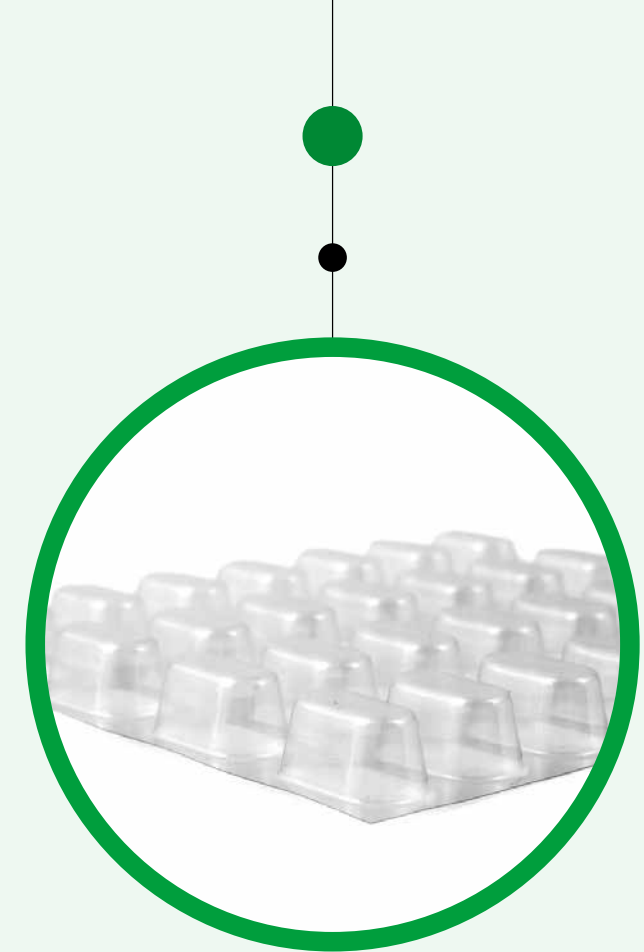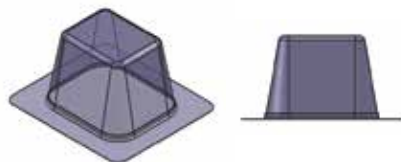


## Key features and benefits

- ✓ Available in a range of different sizes
- ✓ Environmentally friendly material
- ✓ Comply with EU legal regulations
- ✓ Produced by a British retail consortium accredited supplier

If you need advice on which blisters are right for you call us on +44 (0)161 413 5333.

# Custom cards

We understand how important brand and reputation are, which is why we can bespoke our quality packs and print\* your brand on the front cover to promote your pharmacy. We'll talk to you about what you need, create the artwork and then print them exactly as you want them.

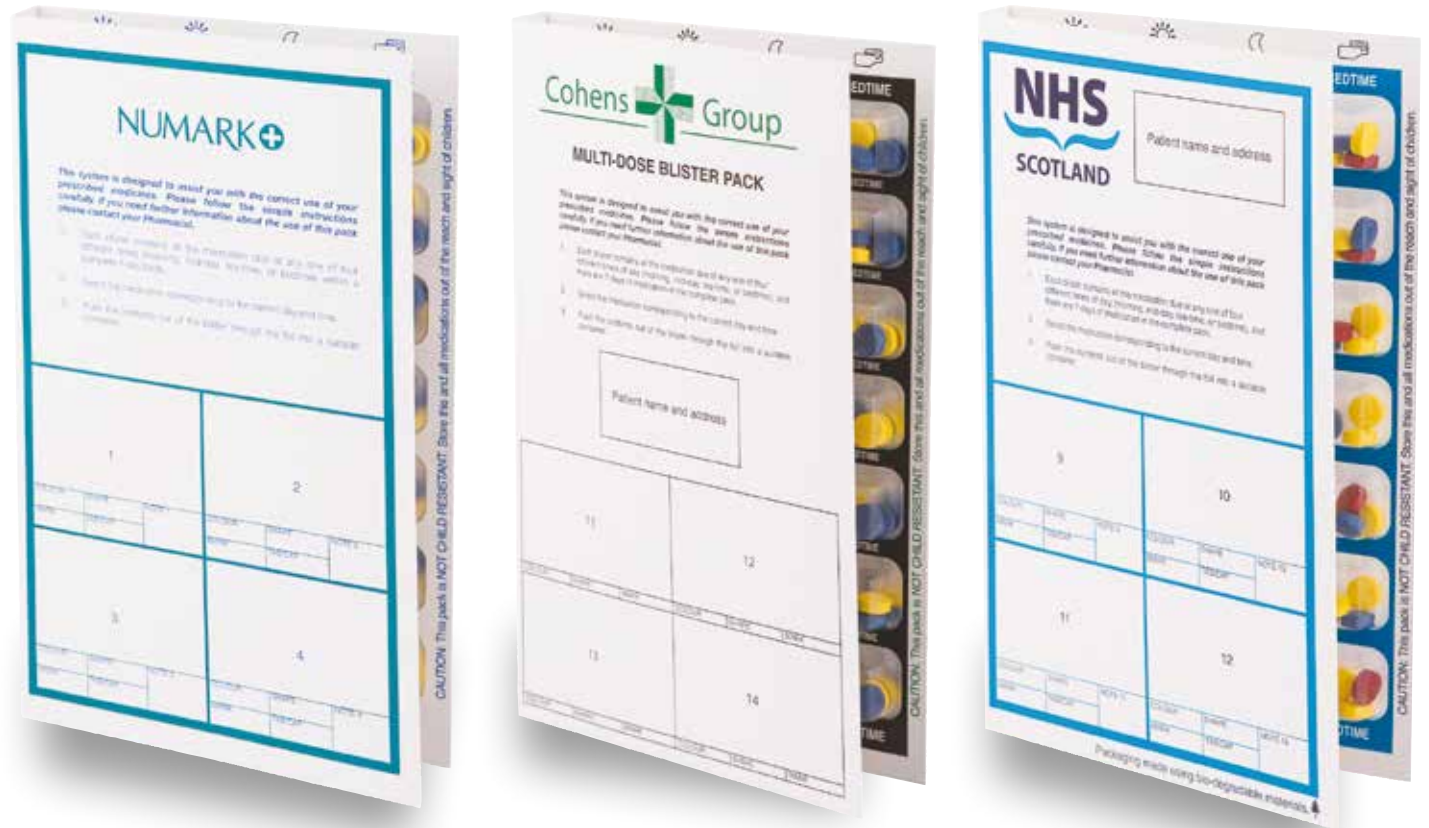

\*minimum order 25,000.

# Accessories for our card range

The following products should be used with our card range of weekly pill packs and monthly care packs.

To make sure you get the best from our pressure seal weekly pill packs and monthly care packs you will need to use a template and roller. Each box of weekly pill packs has a hard filling template inside which is fully recyclable. Rollers can be ordered and you can also order additional templates. To use the heat-seal pill pack you will need a paddle and an easy seal machine (see our technology section).



## Templates

### weekly pill pack

Hard Platen template for our pressure-seal weekly pill packs (excluding flexi).

Qty: 1 unit

Code: 584-64-MP

### duo

Hard Platen template for the pressure-seal weekly duo pack.

Qty: 1 unit

Code: 584-67-MP

### max

Hard Platen template for the pressure-seal weekly pill pack max.

Qty: 1 unit

Code: 584-68-MP

### flexi

Hard Platen template for the pressure-seal weekly pill pack flexi.

Qty: 1 unit

Code: 584-69-MP

### monthly care pack book

Hard Platen template for the pressure-seal monthly care pack book.

Qty: 1 unit

Code: 584-65-MP

### monthly care pack book

Cork template for the pressure-seal monthly care pack book.

Qty: 1 unit

Code: 655-30

### flip

Hard Platen template for the pressure-seal monthly care pack flip.

Qty: 1 unit

Code: 584-65-MP

### heat-seal paddle paddle book

Paddle for heat-seal monthly care pack book.

Qty: 1 unit

Code: 651-31



### large roller

Roller for all card pressure-seal pill packs.

Qty: 1 unit

Code: 660-00



### Pill pack patches

Adhesive pill pack labels. Can be used to change individual doses when patients alter medication.

Qty: 100 patches per roll\*

Code: PLUSPAK-PATCHES

\*2 rolls per pack




### Label flaps


Additional label flaps that can be attached inside the pill packs to allow additional label space.

Qty: 200 sheets per box

Code: 720-LBLFLAPS-EB

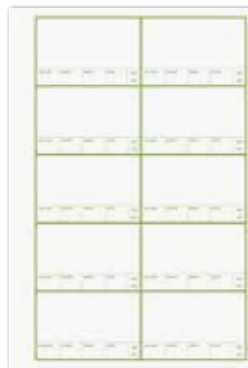

## CARD & CLEAR PACKAGING

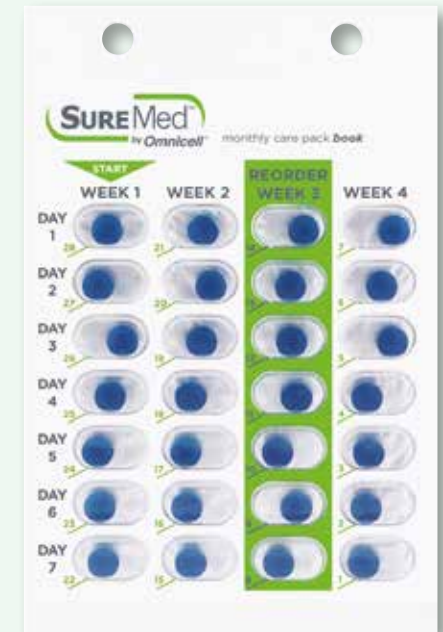

## Safety and reliability

Our card packs are second to none when it comes to the quality of the seal. We glue both sides of the card which means the seal is extremely secure and consistent.

# Our clear range

Our clear range of pill packs are made up of a transparent blister tray and come with seals included.

## weekly pill pack *clear*

With a clear lid and tray

clear pack with blue seals

Qty: 250 units

Code: CL01A

spare backing sheets

Qty: 100 units

Code: CL05

spare seals

Qty: 20 units

Code: CL04

clear pack with clear colour seals


Qty: 250 units

Code: CL01-CSA

clear spare coloured seals

Qty: 20 units

Code: CL04-CS

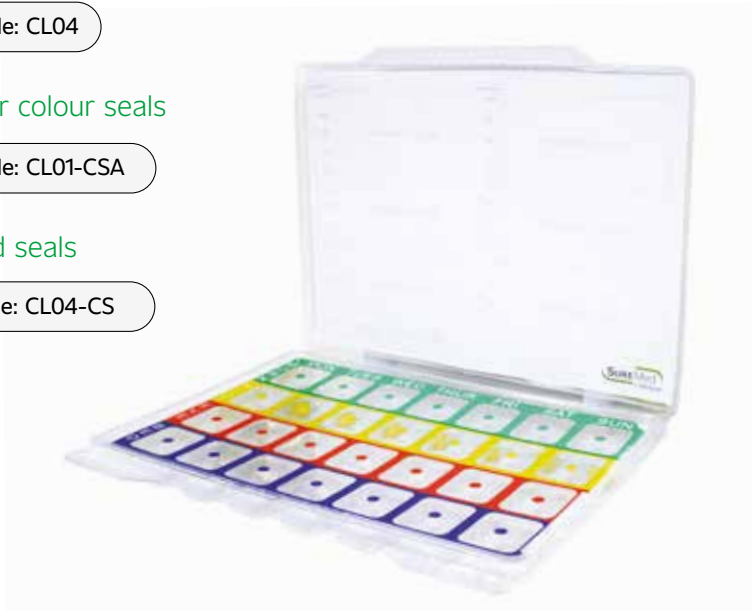

## weekly pill pack *clear max*

Large version of the weekly pill pack clear for patients with more complex medication regimes.

Qty: 100 units

Code: XL01

spare seals

Qty: 100 units

Code: XL01-S

spare backing sheets

Qty: 100 units

Code: XL05





## weekly pill pack *clear duo*

A compact disposable pack for patients on twice daily medication.

Qty: 100 units

Code: N55-D

### spare seals

Qty: 100 units

Code: N85-D

### spare backing sheets

Qty: 100 units

Code: N46-D



## weekly pill pack *clear basic*

Our low cost pill pack. A seven day, four dose pill pack with plastic lid.

Qty: 250 units

Code: CL01-B

Insert and cards

Code: CL02-B



## Weekly pill pack *clear disposable*

MDS disposable tray plus seal, 7 days 6 intakes. Compatible with Suremed cassette

Qty: 250 units

Code: DN14-B

### suremed cassette

Plastic reusable cassette.

Qty: 1 unit

Code: N01

### spare seals

Spare Seals for the weekly pill pack clear disposable

Qty: 20 units

Code: DN15

## extra clear stickers

Can be used with any of the products from the clear range.

Qty: 250 units

Code: CL03

# Monthly care pack clear kit

Suitable for Care Homes.

Our best-selling monthly care pack. The first choice for UK care homes.

## monthly care pack *clear*

Our most popular 28 day pack. A clear tray with intergrated blisters including coloured seals.

### blisters and seals

Qty: 500 units

Code: CN01 and then add: XS, S, M, L, XL

### additional blisters

Qty: 500 units

Code: CN99 and then add: XS, S, M, L, XL

### seals only

Qty: 100 units

Code: CN98 and then add: Y, O, B, P

### sealing template

Qty: 1 unit

Code: CN08A

### small roller

Qty: 1 unit

Code: N27

### antibiotic seal (green)

Qty: 50 units

Code: CNG02

### PRN seal (red)


Qty: 50 units

Code: CNR02

### spare seals (blue)

Qty: 20 units

Code: CN02

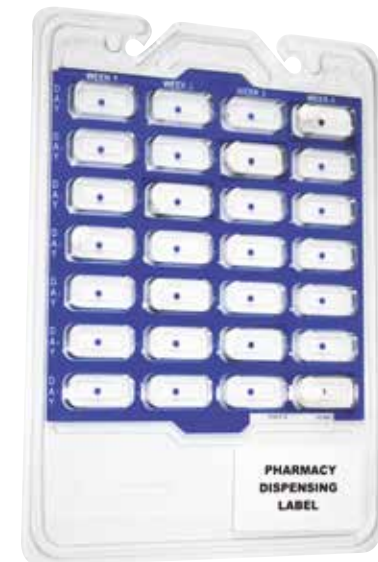

Monthly care pack



Small roller for monthly care pack

**Please note:** Not all of our products feature in our catalogue. If you can't find what you're looking for simply contact us and we'll be able to help.

# Assembling the clear packs

## dividers

Available in a range of colours

Qty: 50 units

Code: CN04  
and then add:

P for pink  
R for red  
Y for yellow  
O for orange  
G for green  
B for blue  
W for white



Monthly care  
pack dividers

## hangers

large hanger

Qty: 1 unit

Code: CN07

## platen

Qty: 1 unit

Code: CL08A




Monthly care  
pack hanger



### 1: Insert

Select a new empty blister tray. Stick the printed SureMed insert for pharmacy labels to the blister in the space provided in the clear tray lid. Place the correct patient medication labels on the instruction flap. Indicate the shape, colour etc. of medication under each label that is applied.

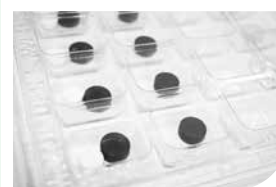

### 2: Dispensing

Select the appropriate medication, dispense the medication into the blister(s). For one tablet place this into the blister, if it is for two tablets then place both into the blister etc. When the blister is filled with the correct dosage of medication, the seal can be fixed to the blister.



### 3: Seal

Carefully peel the backing from the seal, starting from the top right hand corner. Position the seal over the cavities and lower into place. Gently smooth down with a roller to ensure the correct seal.



### 4: Close

When the cold seal is applied to the blister be careful not to pierce any of the seal cavities. Close the clear tray lid and the pack is now complete.

## Do...

- Store the SureMed packs and blisters in their original boxes.
- Store SureMed packs in a dry, indoor and air-conditioned environment.
- Apply firm pressure to pressure seal items.
- Advise the patient how to handle the pack.

## Don't

- Allow the SureMed packs to become damp or wet before, during or after assembly.
- Fold or crease the SureMed packs except where intended along the score lines (Not applicable to clear range)

## Storage advice

We want to make sure our products perform perfectly every time. We always recommend that our SureMed packs should be stored in their original packaging and in a dry environment. Storage temperatures should not exceed the range - 5°C to 37.5°C, be stored in direct sunlight, and should be kept away from sources of heat, and ignition.

If stored properly, our products are expected to provide a quality performance for 24 months from date of production. Production dates are clearly printed on each box. It's important to rotate stock and check storage conditions meet this guidance. Please allow packs to reach room temperature before assembly (approx. 21°C).

# Pill pack and care pack solutions

## Pilbob®

Allows easy removal of medication from pill packs.  
Great solutions for patients with limited dexterity.

Qty: 25 units per pack

Code: 660-10



## Using a Pilbob®

1. Hold the card upright.
2. Move the Pilbob from corner to corner following the edge of the blister.
3. All the medication will fall into the Pilbob and can now be taken.

## Reminder cards

For medication that does not fit on a hanger, the reminder card can be placed on the hangers for ANY of our monthly care packs to ensure a patient's dose is not forgotten.

Qty: 50 units

Code: CN06



## Alert cards

Can be placed on a hanger for ANY of our monthly care packs where a pack has not yet been provided by the pharmacy if the medication was out of stock.

Qty: 50 units

Code: CN18

### Patient profile dividers

For use in the MAR binders.

Qty: 20 units

Code: N98




### MAR sheets

Three part MAR sheets to ensure accurate communication about patient and medication regimes.

Qty: 750 units

Code: L-MAR-3

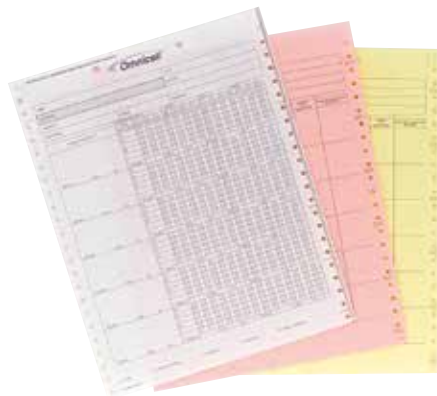

### Single blank cassette medication card

Qty: 750 units

Code: N13-900707

## CARD & CLEAR PACKAGING



### Binders

Large ring binder to hold MAR sheets.

Qty: 1 unit

Code: BINDERS-THK

Standard ring binder to hold MAR sheets.

Qty: 1 unit

Code: BINDERS



### Patient profile sticker

Patient labels which can be affixed to the dividers on ANY of our monthly care packs.

Qty: 250 units

Code: CN05

### Patient medication time stickers

Can be affixed to top divider used with monthly care packs.

Qty: 50 units

Code: CN19

# Pill pack and care pack solutions

## Medication update pad

Qty: 1 unit

Code: N39-A



## Paper medication cups

10Z paper souffle pots can be used when administering medication.

Qty: 5000 per box

Code: 670-03



## Tag bag

Small, clear resealable bag.

Qty: 500

Code: N40

## Tag bag returns form pad

Qty: 5

Code: N96

## Tag bag label

Labels for resealable tag bags.

Qty: 500

Code: N41

## White tag seals

Qty: 100

Code: N66



## Black security tags

Qty: 100

Code: N66-A





ellizine® CR 200  
ellizine® CR 200

52  
50

32  
3

59

# Stickers and labels



ON LEAVE  
Stickers

Qty: 250

Code: CN11



NAUSEA  
Stickers

Qty: 250

Code: CN13



HOSPITAL  
Stickers

Qty: 250

Code: CN14



ASLEEP  
Stickers

Qty: 250

Code: CN15



REFUSED DOSE  
Stickers

Qty: 250

Code: CN09



CLINICAL REASONS  
Stickers

Qty: 250

Code: CN12

## Sticker pack

Includes the following stickers:

- On leave
- Nausea
- Hospital
- Asleep
- Refused dose
- Clinical reasons

Qty: 6 rolls

Code: CN16



## Blank Thermal

1 roll = 3000 stickers

Size 1: 6"x8"

Qty: 1

Code: L-201





Security seals

Qty: 100

Code: N31



Coloured spot stickers

Qty: 1

Code: N42



Cassette owner sticker

Qty: 100

Code: N38



Weekly labels

Qty: 250

Code: L-902



Daily labels

Qty: 250

Code: L-903




Unprinted dose int sticker

Blank dose interval stickers.

Qty: 100

Code: N37

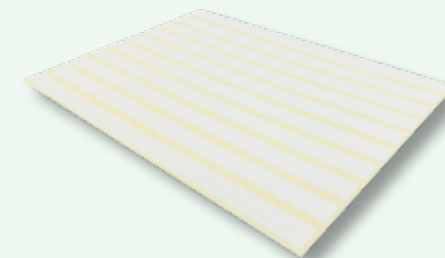

Community patient profile labels

Qty: 100

Code: N64



Tray ID label

Qty: 100

Code: N61



SureMed by Omnicell

# De-blistering solutions

DB500 de-blistering machine

## DB500 de-blistering machine

De-blister up to 280 tablets per minute with the DB500 de-blistering machine!

- ✓ The fastest manual de-blisterer available
- ✓ De-blisters the majority of tablets and capsules
- ✓ No more repetitive strain injury or sore fingers
- ✓ Quick calibration, ideal for de-blistering even small quantities
- ✓ A must for pharmacies offering adherence support to care homes or individual patients

Qty: 1 unit

Code: 100-62M

Spare DB500 tray

Qty: 1 unit

Code: 100-64M




# FREE weekly pill pack point of sales toolkit

If you're helping your patients by offering a weekly pill pack service, we have a digital point of sales toolkit you can download for FREE.

Email [marketinguk@omnicell.com](mailto:marketinguk@omnicell.com) and we'll give you access.

Scan here to find out more:

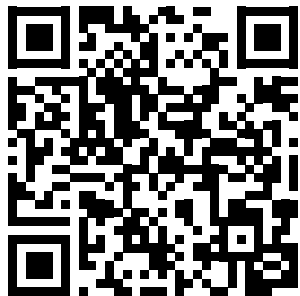

You can add your pharmacy name to the items in the pack which includes:

- A5 patient leaflet
- Content and images to publish on your website
- A5 leaflet to inform your care home customers that you are now offering this service
- A3 patient poster
- Social media posts and images to share across your pharmacy's channels



# Community pharmacy automation from Omnicell

As well as being the UK's preferred medication adherence packaging brand, Omnicell is a world leader in community pharmacy automation solutions.

Our solutions help to reduce medication errors, improve patient safety and streamline operations so that clinical and care staff can spend more time on face to face patient care.



You can request more information about our pharmacy automation solutions here.

Omnicell offers the complete solution to automate all your pharmacy workflows.

## Prescription collection

Offer your patients the convenience of prescription collection whenever it's most convenient for them, 24/7, with the Pharmaself24 prescription collection point. A perfect differentiator to increase your patient numbers.

## Medication pack-filling automation (MDS)

Produce up to 40 sealed and checked MDS cards per hour, regardless of the number of pills per blister with the VBM 200F. Ideal for pharmacies looking to scale their MDS business. At a lower price point are the ultra-compact blister and pouch filling solutions from JVM, ideal for pharmacies with up to 300 MDS patients.

## Stock management & pack handling

Let the Medimat Robotic Dispensing system sort, load and store your medication packs, and dispense them on-demand to the counter. Bring cost, time and space efficiency to your pharmacy operation.



**Pharmaself24 Prescription  
Collection Point**



**MDS Pack  
Automation**



**Medimat Robotic  
Dispensing System**

## How to order

Call: +44 (0)161 413 5333

Email: [ordersuk@omnicell.com](mailto:ordersuk@omnicell.com)

Visit: [www.Omnicell.co.uk](http://www.Omnicell.co.uk)



Omnicell and the Omnicell design mark are registered trademarks and XT is a trademark of Omnicell, Inc. in the United States and internationally. All other trademarks and trade names are the property of their respective owners. ©2024 All rights reserved.